



**Optional Extra's**

- o Adaptor (ADT)



**Description:**

Our pressure mat is designed to monitor a service user in a variety of situations –

- o The most popular approach is for alerting staff to night-time, e.g. Dementia patients leaving the security of their bedroom.
- o Should a patient who has balance problems begin to leave a chair or bed unaided then an assistance call can be created to alert staff to the problem
- o Frail patients attempting to get out of a chair unaided can also be monitored so that an assistance call is generated if they try to leave the chair.

The pressure mat can be used with the ADT adaptor to work in conjunction with an existing ARM Pear Push unit (NPP) in the standard configuration i.e. where the client steps onto the mat to activate it.

If used in the reverse configuration where the client gets off the mat to trigger an assistance call then a new pear push lead and an updated connection socket for the ARM Pear Push unit are required, to enable the use of the existing bedside pear push unit.

This is available as a kit complete with the pressure mat, order part number is REV-PRM-KIT

Care should be taken with the use of pressure mats so that the cable is always routed under rugs and away from the service users to minimise trip hazards.

**Details:**

- o Dimensions 860 x 540mm (Approx 33¾ x 21¼")
- o Wipe clean surface

In pursuance of a policy of continued improvement, the company reserves the right to change the design and specification without prior notice

Supplier:

Pressure Mat

Alarm Radio Monitoring Ltd  
Brunel Road  
Enterprise Park  
Leominster  
HR6 0LX

Tel: 01568 610016  
Fax: 01568 615511

**E-mail:**  
[sales@arm.uk.com](mailto:sales@arm.uk.com)

**Website:**  
[www.arm.uk.com](http://www.arm.uk.com)